Fluimar

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chemist's research

Fluimar

Sterile sea water solution
Daily nasal hygiene

Fluimar Nasal hygiene, a natural product, is a solution in unidose, naturally rich in mineral salts and marine trace elements. It can be used on a daily basis due to its physiological concentration (isotonic - 9%).

Unidoses **Fluimar Nasal hygiene** are recommended for infants (except premature babies), children and adults for daily cleaning of the nasal cavity.

COMPOSITION: Sea water, purified water (equivalent to 9 g/L of salts).

PRESENTATION: Box of 20 x 5mL unidose containers.

INDICATIONS: **Fluimar Nasal hygiene** is recommended for infants from birth (except premature babies), children and adults as an instillation or wash:

- For daily nasal hygiene
- It hydrates the nasal mucosa
- It facilitates the evacuation of nasal secretions
- It is recommended in case of allergic rhinitis.

DOSAGE: 1 to 4 times a day, in each nostril as necessary.

INSTRUCTIONS FOR USE: The presentation and the unidose shape allow easy and safe use.

- It is recommended for an infant to lie down, for a child to sit or stand with the head tilted to the side.
- Detach an unidose and open it by turning the top part.
- Gently insert the tip into the nostril and press the unidose gently.
- Repeat the operation with the other nostril.
- Straighten the head to allow the mucus to drain out.
- For infants: wipe off the excess. Use a baby fly in case of strong discharge in order to gently suck up the mucus.
- In young children: ask them to blow their nose gently.

PRECAUTIONS FOR USE: For infants, instill the product with a minimum of pressure to avoid contamination of the middle ear.

CONTRAINDICATIONS: Do not use in case of hypersensitivity to seawater. Do not use on premature babies.

WARNINGS: External use. Do not inject. Do not swallow. Single use.

Do not use an already opened or damaged unidose due to the risk of rapid bacterial contamination.

If symptoms persist or worsen, see your doctor.

Please report any serious incidents relating to this device to Laboratoire Gilbert - website:

www.labogilbert.fr and/or to the national Competent Authority.

KEEP OUT OF SIGHT AND OUT OF REACH OF CHILDREN.

STORAGE CONDITIONS: Do not exceed the expiry date indicated on the unidose. Keep away from light.

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Chemist's Research S.r.l.